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•

<b>SILVERLON</b> <i>PERFORMANCE. POWER. SPEED.</i>

•

•

# Silverlon® Acute Burn Glove Dressings

**Publication Date: 10/2023**

					
---	---	---	---	---	---

	
<b>Emergo Europe, Westervoortsejdijk 60; 6827 AT Arnhem The Netherlands Tel. +(31) (0) 70 345-8570</b>	<b>Argentum Medical, LLC 2571 Kaneville Ct Geneva, IL 60134 USA Tel. (+1) 888.551.0188 Fax. (+1) 888.558.9923 www.silverlon.com</b>

LC-CE-IFU-ABG-03; Rev C

<b>SILVERLON</b> <i>PERFORMANCE. POWER. SPEED.</i>	Silverlon® Acute Burn Glove Dressings	English
<p><b>Device Description</b></p>	<p>Silverlon® Acute Burn Glove Dressings are a sterile, single layer, non-aderent, antimicrobial barrier dressing in the form of a glove with open finger, which is designed to fit comfortably over the hand, covering the entire skin surface of the hand.</p>	

<b>Available as:</b>	
<b>Code</b>	<b>Size</b>
ABG-01XS	Adult Extra Small
ABG-01S	Adult Small
ABG-01M	Adult Medium
ABG-01L	Adult Large
ABG-01XL	Adult Extra Large
ABG-01XXL	Adult Extra Extra Large

Silverlon® Acute Burn Glove Dressing contains silver to provide an ancillary, local antimicrobial effect to reduce the risk of wound infection by organisms sensitive to silver. Ionic silver is a broad spectrum antimicrobial, which has been shown to be effective against a range of pathogens associated with wound infection. Silverlon® dressings have been tested in vitro and found effective against microorganisms such as: Enterococcus faecalis, Staphylococcus aureus, Pseudomonas aeruginosa, Acinetobacter Baumannii, Enterobacter cloacae, Staphylococcus epidermidis and Klebsiella pneumoniae.

In vitro testing has shown Silverlon® dressings to be MRI compatible.

**Indications**
Silverlon® Acute Burn Glove Dressing is a professional use wound dressing intended for local management of the following types of hand wounds:

- 1st and 2nd degree burns
- Donor and graft site wounds
- Traumatic and surgical wounds
- Infected wounds\*

\*Silverlon® Acute Burn Glove Dressing may be used on those infected wound types listed above. Where used on infected wounds, the infection should be treated as per local clinical protocol.

Silverlon® Acute Burn Glove Dressing is indicated for a total duration of contact (i.e. where treatment involves consecutive application of individual dressings) of up to 30 days.

**Warnings**

- Do not use past expiration date on the product packaging.
- Do not use if pouch is damaged or open.
- Clinicians / Healthcare Professionals should be aware that there are very limited data on prolonged and repeated use of silver-containing dressings, particularly in children and neonates.
- Do not use petroleum-based ointments or creams under Silverlon® Acute Burn Glove Dressing.
- Do not moisten Silverlon® Acute Burn Glove Dressing with hydrogen peroxide or povidone iodine.
- Silverlon® Acute Burn Glove Dressings are intended for single use only and should not be re-used. Re-use may adversely affect the dressing’s performance characteristics and also presents a risk of infection to the patient.
- Do not remove Silverlon® Acute Burn Glove Dressing from the hand while dry or if the dressing sticks when trying to remove it. Follow the instructions below for dressing removal.
- Total duration of use of Silverlon® Acute Burn Glove Dressing (i.e. where treatment involves consecutive application of individual dressings) should not exceed 30 days.

**Contraindications**

- Avoid using Silverlon® Acute Burn Glove Dressing on patients with known sensitivity to silver or nylon.

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