**SILVERLON**

PERFORMANCE. POWER. SPEED.

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**SILVERLON**

PERFORMANCE. POWER. SPEED.

## Silverlon® Lifesaver® IV/Catheter Dressings

**English**

### Device Description

Silverlon® Lifesaver® IV/ Catheter Dressings are sterile, multi-layer, non-adherent, absorbent, antimicrobial barrier dressings, intended for topical use in wound management, to reduce risk of wound infection.

### Available as:

Code	Pad Size
Lifesaver™ Ag IV/CD-15	2.5 cm circle with 1.5 mm center hole
Lifesaver™ Ag IV/CD-40	2.5 cm circle with 4.0 mm center hole
Lifesaver™ Ag IV/CD-70	2.5 cm circle with 7.0 mm center hole

Silverlon® Lifesaver® IV/ Catheter Dressings contain silver to provide an ancillary, local antimicrobial effect to reduce the risk of wound infection by organisms sensitive to silver. Ionic silver is a broad spectrum antimicrobial, which has been shown to be effective against a range of pathogens associated with wound infection. Silverlon® dressings have been tested in vitro and found effective against microorganisms such as: Enterococcus faecalis, Staphylococcus aureus, Pseudomonas aeruginosa, Acinetobacter Baumannii, Enterobacter cloacae, Staphylococcus epidermidis, and Klebsiella pneumoniae.

In vitro testing has shown Silverlon® dressings to be MRI compatible.

### Indications

Silverlon® Lifesaver® IV/ Catheter Dressings are professional use wound dressings indicated for local management of:

- Vascular access sites (including around IV catheters, central venous lines, arterial catheters, dialysis catheters);
- Drainage tube access sites;
- Epidural catheter access sites;
- Infected Wounds\*

\*Silverlon® Lifesaver® IV/Catheter Dressings may be used on those infected wound types listed above. Where used on infected wounds, the infection should be treated as per local clinical protocol.

Silverlon® Lifesaver® IV/ Catheter Dressings are indicated for a total duration of contact (i.e. where treatment involves consecutive application of individual dressings) of up to 30 days.

### Warnings

- Do not use past expiration date on the product packaging.
- Do not use if pouch is damaged or open.
- Clinicians / Healthcare Professionals should be aware that there are very limited data on the prolonged and repeated use of silver-containing dressings, particularly in children and neonates.
- Silverlon® dressings should not be used during pregnancy or lactation because of the lack of data concerning the effects of silver on reproduction.

**SILVERLON**

PERFORMANCE. POWER. SPEED.

# Silverlon® Lifesaver® IV/Catheter Dressings

EC REP

Emergo Europe,  
Prinsessegracht 20,  
2514 AP The Hague,  
Netherlands  
Tel.: +(31) (0) 70.345.8570

LC-CE-IFU-IVCD; Rev 08

Argentum Medical  
2571 Kaneville Court  
Geneva, IL 60134 USA  
Tel. (+1) 888.551.0188  
Fax. (+1) 888.558.9923  
[www.silverlon.com](http://www.silverlon.com)