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**English**

## Device Description

Silverlon® Wound Contact Dressing, Burn Contact Dressing, Elastic Burn Wrap and Digit Sleeve Dressings are sterile, single layer, non-adherent, antimicrobial barrier dressings, intended for topical use in wound management, to reduce risk of wound infection.

## Available as:

Code	Size	Code	Size	Code	Size
WCD22	5 x 5 cm	BCD44	10 x 12 cm*	BCD-CDM	66 x 64 cm
WCD44	10 x 12 cm	BCD48	10 x 20 cm	BCD-CDL	76 x 65 cm
WCD412	10 x 30 cm	BCD816	20 x 40 cm	BWD466	10 x 168 cm
WCD1012	25 x 30 cm	BCD1616	40 x 40 cm	BWD6108	15 x 274 cm
WCD466	10 x 168cm	BCD2424	61 x 61 cm	DS112	3 x 30 cm

Silverlon® Wound Contact Dressing, Burn Contact Dressing, Elastic Burn Wrap and Digit Silverlon® Wound Contact Dressing and Burn Contact Dressing contain silver to provide an ancillary, local antimicrobial effect to reduce the risk of wound infection by organisms sensitive to silver. Ionic silver is a broad spectrum antimicrobial, which has been shown to be effective against a range of pathogens associated with wound infection. Silverlon® Wound Contact Dressing and Burn Contact Dressing have been tested in vitro and found to be effective against micro organisms such as: Enterococcus faecalis, Staphylococcus aureus, Pseudomonas aeruginosa, Acinetobacter Baumannii, Enterobacter cloacae, Staphylococcus epidermidis, and Klebsiella pneumoniae.

In vitro testing has shown Silverlon® dressings to be MRI-compatible.

## Indications

Silverlon® Wound Contact Dressing, Burn Contact Dressing, Elastic Burn Wrap and Digit Sleeve Dressing are professional use wound dressings indicated for local management of:

- 1st and 2nd degree burns
- Donor and graft site wounds
- Traumatic and surgical wounds
- Diabetic, pressure and venous ulcers
- Infected wounds\*

\*Silverlon® Wound Contact Dressing, Burn Contact Dressing, Elastic Burn Wrap and Digit Sleeve Dressing may be used on those infected wound types listed above. Where used on infected wounds, the infection should be treated as per local clinical protocol.

Silverlon® Wound Contact Dressing, Burn Contact Dressing, Elastic Burn Wrap and Digit Sleeve Dressing are indicated for a total duration of contact (i.e. where treatment involves consecutive application of individual dressings) of up to 30 days.

## Warnings

- Do not use past expiration date on the product packaging.
- Do not use if pouch is damaged or open.
- Clinicians / Healthcare Professionals should be aware that there are very limited data on prolonged and repeated use of silver-containing dressings, particularly in children and neonates.
- Silverlon® Wound Contact Dressing, Burn Contact Dressing, Elastic Burn Wrap and Digit Sleeve Dressing should not be used during pregnancy or lactation because of the lack of data concerning the effects of silver on reproduction.
- Do not use petroleum-based ointments or creams under Silverlon® Wound Contact Dressing, Burn Contact Dressing, Elastic Burn Wrap and Digit Sleeve Dressing.
- Do not moisten Silverlon® Wound Contact Dressing, Burn Contact Dressing, Elastic Burn Wrap and Digit Sleeve Dressing with hydrogen peroxide or povidone iodine.
- Silverlon® Wound Contact Dressing, Burn Contact Dressing, Elastic Burn Wrap and Digit Sleeve Dressing are intended for single use only and should not be reused. Re-use may adversely affect the dressing's performance characteristics and also presents a risk of infection to the patient.
- Total duration of use of Silverlon® Wound Contact Dressing, Burn Contact Dressing, Elastic Burn Wrap and Digit Sleeve Dressing (i.e. where treatment involves consecutive application of individual dressings) should not exceed 30 days.

**Compressas Silverlon® Wound Contact**  
**Compressas Silverlon® Burn Contact**  
**Compressas Silverlon® Elastic Burn Wrap**  
**Compressas Silverlon® Digit Sleeve**

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