

# Anasept® Case Study 3

## Leg Wound, Clinical Case Study by: Jean O. Galloway, PT-Superior Rehab Center Newport Beach, CA.

### ***Clinical Challenge:***

To heal wound on Right leg.

### ***The Patient:***

Mrs. N. is an 89 year old female who struck her right anterior lateral leg on the corner of her car door. It did not break the skin on impact. Incident occurred on 2/6/2014.

### ***Start of Claim:***

I first saw the patient on 3/6/2014. Wound measured 1.8 x 1.7 (see photo below). Tissue was dark red and grey. Patient said that there had been a bruised area for several days. It then developed into a hematoma. Patient saw her physician for 3 or 4 times and was told to use Neosporin, however, it continued to draining so she was referred to SRC. I debrided and decided to use the Anasept Gel. The Anasept spray is used prior and after debriding. I covered with a Primopore and asked to see patient twice weekly.

### ***Continued Treatment:***

Patient continued to be seen twice weekly with debriding and Anasept Gel the protocol. Wound looked cleaner and smaller with each visit. On 3/20/2014 wound now measured 0.8x 0.3. I was extremely pleased with the quick results the patient was achieving.

New photo was taken on 3/28/2014 and wound was closed and healed (see photo below). Patient was discharged and delighted.

### ***Clinical/Results Statement:***

Dramatic evident as the patient was seen only 7 times and wound was closed and healed. I attribute the use of the Anasept Gel as the reason for this. I am quite pleased and will continue the use of this product.



3/6/2014 wound measured 1.8 x 1.7



3/28/2014 wound was closed and healed.

Articles written and photos taken by:  
Jean O. Galloway, PT  
Superior Rehab Center, Inc.  
1441 Superior Ave, Suite B  
Newport Beach, CA 92663