



Antimicrobial Skin And Wound Gel

Hospital and Professional use

Reorder Information, Catalog #5015G

PRODUCT DESCRIPTION:

Anasept® Antimicrobial Skin and Wound Gel is a clear, amorphous, isotonic hydrogel that helps maintain moist wound environment that is conducive to healing, by either absorbing wound exudate or donating moisture while delivering 0.057% broad spectrum antimicrobial sodium hypochlorite. Anasept Gel inhibits the growth of microorganisms, such as *Acinetobacter baumannii*, *Clostridium difficile*, *Escherichia coli*, *Proteus mirabilis*, *Pseudomonas aeruginosa*, *Serratia marcescens*, *Staphylococcus aureus*, Carbapenem Resistant *Escherichia coli* (CRE), Vancomycin Resistant *Enterococcus faecalis* (VRE), and Methicillin Resistant *Staphylococcus aureus* (MRSA), that are commonly found in wound bed, as well as, fungi such as *Aspergillus niger*, *Candida albicans*, and *Candida auris*.

INDICATIONS:

Anasept Gel is intended for OTC use for management of skin abrasions, minor irritation, lacerations, cuts, exit sites and intact skin.

Professional Use: Anasept Gel is intended to be used under the supervision of a healthcare professional in the management of wounds such as stage I-IV pressure ulcers, partial & full thickness wounds, diabetic foot & leg ulcers, post-surgical wounds, first & second degree burns, grafted & donor sites.

DIRECTIONS FOR USE:

SKIN:

- 1) Cleanse site with appropriate skin cleanser.
- 2) Apply thin layer of Anasept Gel to intended site. Repeat as necessary.

WOUND:

- 1) Cleanse and/or debride wound as necessary.
- 2) Apply generous amount of Anasept Gel directly onto wound bed. (1/8" to 3/16" thick).
- 3) Cover the wound with sterile gauze or other appropriate wound dressing and secure in place. (Avoid wound dressings containing silver and other metals). Maintain a moist wound environment between dressing changes.

Store at Room Temperature up to 25°C (77°F)

EXTERNAL USE ONLY. NOT FOR OPHTHALMIC USE.

Discontinue use if redness or irritation occurs.

latex FREE

Manufactured in the USA by:
 Argentum Medical, LLC.
 2571 Kaneville Court
 Geneva, IL 60134
 (888) 551-0188

www.anacapa-tech.net

Anasept® is a registered trademark of Argentum Medical, LLC.

Protected by US and Foreign Patents.

APPROVAL LABEL DICR# 23-004
 Doc# LC-IFU-ANA-SEPT Rev# A
 Operations: [Signature] Date: 10 JAN 2023
 Quality Assurance: [Signature] Date: 10 JAN 2023
 Customer / Other: [Signature] Date: 10 JAN 2023

RP
 JAN 23 2023



Argentum Medical

LC-IFU-ANA-SEPT
 Rev A