

# SILVER-SEPT®

Silver Antimicrobial Skin & Wound Gel

## Reorder Information, Catalog #3015S (1.5oz Tube)

### PRODUCT DESCRIPTION:

Silver-Sept® Silver Antimicrobial Skin & Wound Gel, containing 200µg/gram total silver, is a clear, amorphous hydrogel wound dressing that helps to maintain a moist wound environment that is conducive to healing. Silver-Sept® will not stain or discolor tissue.

Silver-Sept® Gel functions as a long-lasting antimicrobial barrier by inhibiting the growth of common bacteria such as: *Escherichia coli*, *Proteus mirabilis*, *Pseudomonas aeruginosa*, *Serratia marcescens*, *Staphylococcus aureus*, *Methicillin resistant Staphylococcus aureus (MRSA)*, and *Vancomycin resistant Enterococcus faecalis (VRE)*, as well as fungi such as, *Aspergillus niger* and *Candida albicans*.

### INDICATIONS:

Silver-Sept® Gel is intended for OTC use for Abrasions & Lacerations and under the supervision of a healthcare professional in the management of:

- Stage I-IV Pressure Ulcers
- Partial and Full Thickness Wounds
- 1st and 2nd degree burns
- Diabetic Foot and Leg Ulcers
- Grafted and Donor Sites

### DIRECTIONS FOR USE:

- 1) Cleanse or debride wound as necessary
- 2) Apply a generous amount of Silver-Sept® Gel directly onto wound bed (1/8" to 3/16" Thick)
- 3) Cover with a sterile gauze or other appropriate secondary dressing and secure in place.
- 4) Maintain a moist wound environment between dressing changes.

**NOTE:** Silver-Sept® Silver Antimicrobial Skin & Wound Gel may remain in the wound bed for up to 3 days. More frequent dressing changes may be required dependent upon the amount of wound exudate present and the condition of the secondary dressing.

**EXTERNAL USE ONLY. NOT FOR OPHTHALMIC USE.**

**Store at room temperature: 15°-30°C (59°-86°F).**

## latex FREE

Manufactured in the USA by:

Argentum Medical, LLC.

2571 Kaneville Court

Geneva, IL 60134

(888) 551-0188

[www.anacapa-tech.net](http://www.anacapa-tech.net)

Silver-Sept® is a registered trademark of Argentum Medical, LLC.



LC-IFU-SIL-SEPT  
Rev B

APPROVAL LABEL DICR# 23-145  
Doc# LC-IFU-SIL-SEPT Rev# B  
Operations Date 14 NOV 2023  
Quality Assurance Date 14 NOV 2023  
Customer / Other Dept Date 15 Nov 2023  
NOV 16 2023