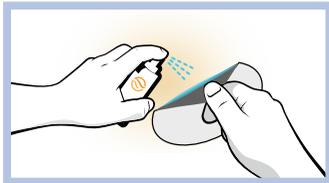
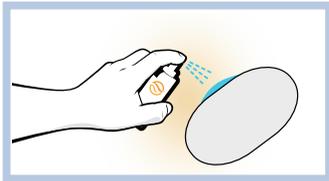
- Removes hydrocolloid on contact
- Helps remove tapes and bandages
- Quick drying – no residue

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HOW TO USE

Instantly removes hydrocolloid, dressing tapes and other adhesives without irritation. Flash dries in 15 seconds without leaving a residue which allows for immediate reapplication.

Hold foil packet vertically, tear at notch to open and remove cloth from packet. Keep cloth folded to retain maximum saturation.

! **Keep the packet held upright to avoid spilling the excess FreeDerm solution.**

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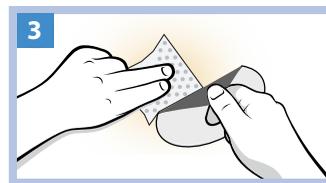
DIRECTIONS: For external use only on normal, intact skin. Spray edges of adhesive until wet. Gently lift the adhesive from the skin. While lifting, spray the exposed adhesive until completely released. If redness or irritation occurs, discontinue use and consult your healthcare professional.

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
Drip the excess FreeDerm from the packet onto the top edge of the dressing, allowing the solution to drip downwards to start releasing the adhesive bond between the skin and dressing.

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Once the dressing edge is lifted, use the folded cloth to wipe underneath the dressing as you gently pull the dressing off.

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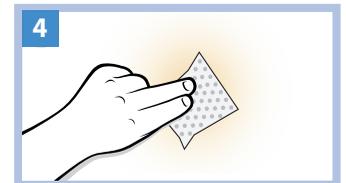Once the dressing is completely removed, wipe the area to remove any residue adhesive.

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WARNING! Highly Flammable. Avoid contact with eyes and mouth. For external use only. Avoid inhalation.

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Manufactured for: BioDerm, Inc. 12320 73rd Ct. North, Largo, FL 33773, USA
Fabriqué pour : BioDerm, Inc. 12320 73rd Ct. North, Largo, FL 33773, USA (États-Unis)